

Package leaflet: Instructions for use

PF DROPS[®]

Sodium Chloride 5%

Preservative free

Sodium Chloride 5% hypertonic eye drops - preservative free 10ml bottle

Ingredients

Sodium chloride, Sodium edetate, Sodium hyaluronate, Boric acid, Sodium tetraborate, Water.

What is PF Drops Sodium Chloride 5%?

PF Drops Sodium Chloride 5% are preservative free and supplied in a 10ml bottle equipped with the innovative and patented multi-dose Ophthalmic Squeeze Dispenser (OSD).

The OSD keeps potentially harmful bacteria away from the solution, which remains sterile for up to 2 months after first use.

When should PF Drops Sodium Chloride 5% be used?


PF Drops Sodium Chloride 5% is a hypertonic ophthalmic solution indicated for temporary alleviation of corneal oedema symptoms.

How to use PF Drops Sodium Chloride 5%?

Remove the tamper-evident seal on first use. Remove the bottle cap to prevent drying of the tip before instilling 1 drop into the eye. Put the cap back on the bottle after each use.

Repeat every 3 or 4 hours or as prescribed by your doctor.

Use the drops at least once daily to keep the nozzle of the bottle clear.

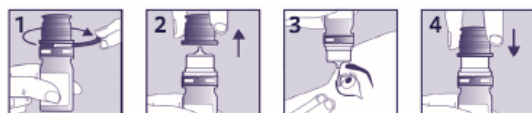

Precautions and warnings

- Do not use if the container or packaging is damaged.
- Do not use if you are sensitive to any of the components of the product.
- Do not use after the expiry date shown on the carton and container.
- Do not touch the eye with the tip of the container.
- The use of the product may induce burning and temporary irritation. In case the problem persists or in case of ocular irritation, please consult your doctor.
- If you wear soft contact lenses, remove them before applying the product and do not replace the lenses for at least 15 minutes.
- Wait at least 10-15 minutes before instilling any other ophthalmic drug.
- After opening the bottle, use the product within 2 months.
- Keep out of the sight and reach of children.
- Do not use in presence of quaternary ammonium salts.
- Do not swallow.
- This product is not normally intended for use for more than 30 days; if the symptoms persist consult your doctor.



STERILE A



NTC s.r.l.

Via Luigi Razza 3 • 20124 • Milan • Italy

Distributed in the UK and IE by:

Aspire Pharma Ltd.,
Unit 4, Rotherbrook Court, Bedford Rd,
Petersfield, Hampshire, GU32 3QG, UK
Revision 12/2019

1010432-P4.2 LFSOD01801

ASPIRE[®]
P H A R M A